

VistaScan View wall mounting set up

The following information is supplementary to the appliance Installation and Operating Instructions.


Always refer to the Installation and Operating Instructions of the appropriate appliance in which important information is detailed, e. g. Safety Instructions, Set-up, Electrical Connections, Disinfection, Cleaning, etc.

Delivery Contents


- Wall mounting bracket 2151-001-00
 - Wall bracket
 - Knurled screw (3x)



1.

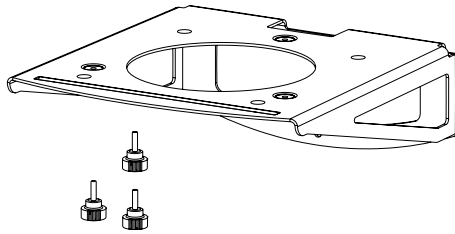

2.

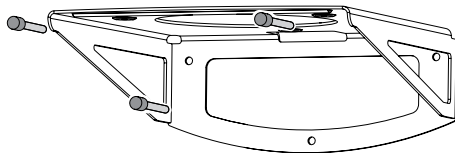


- Place the wall mounting bracket in place horizontally and secure using appropriate materials.

3.


- Place the VistaScan View onto the wall mounting bracket and line up with the drilled holes.

4.

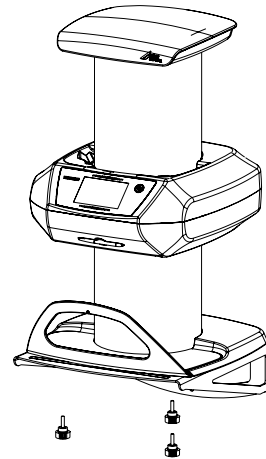

- Use the knurled screws to fix the VistaScan View securely onto the wall mounting bracket.

5.

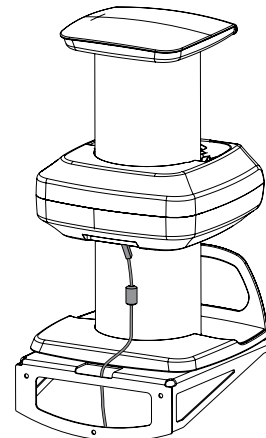

- Feed the cable through the cable opening.